

Step into the Future of Digital Casting

Step 1: **Get signed up on the eLab** by providing an email address and your desired user name.

Email: _____

Desired User Name: _____

Account #: _____ - _____ - _____



Step 2: **Acquire an iPad and a Structure Sensor Scanner.** Visit structure.io (that's their website) to learn more about the Structure Sensor Scanner and to purchase the correct version to fit and function with your iPad.

Digital Casting

Step 3: **Download KLM's Digital Casting Application.** Using your iPad, login at elab.klmlabs.com and find step "2" on the home page. Click the download link and follow the directions... You will need to "Trust" the application by going on your iPad to Settings > General > Device Management > KLM OTHOTIC LABORATORIES INC > Trust. Lastly enter your eLab user name in the Settings of the Digital Casting Application. This will link the App with your eLab account ensuring your Digital Casts upload correctly.

Step 4: Watch the Digital Casting instructional videos at www.klmlabs.com/videos

Step 5: **Start Casting!** KLM will accept three initial orders with Digital Casts, each requiring both your approval as well as the approval of KLM's Digital Casting Department, before opening your account to submit Digital Casts with no restrictions. This is to ensure the accuracy of your product. KLM is proud to manufacture quality custom devices. With respect to all parties, we request you participate in this "testing" period to ensure your success with this new technique.

Please complete this form and email to cservice@klmlabs.com